

# share data with your clinic

**dexcom**  
CLARITY


You can authorize data sharing with your clinic directly from your Dexcom G7 app. Your care team will then have access to your information during visits or anytime you might need assistance.

1 Open your Dexcom G7 app



Dexcom G7


2 Enter clinic code: \_\_\_\_\_

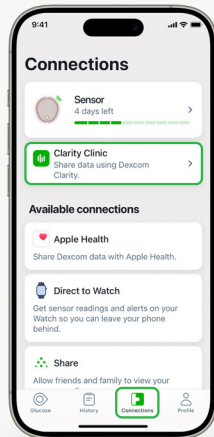


Select Connections & click Clarity Clinic



Select Continue

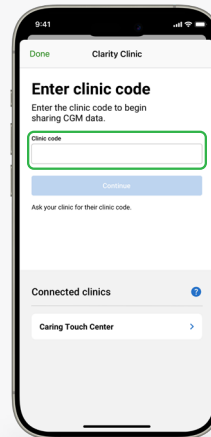

Enter your clinic code and select Continue

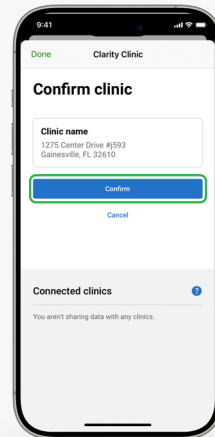

Confirm your clinic

**BRIEF SAFETY STATEMENT:** Failure to use the Dexcom Continuous Glucose Monitoring System and its components according to the instructions for use provided with your device and available at <https://www.dexcom.com/safety-information> and to properly consider all indications, contraindications, warnings, precautions, and cautions in those instructions for use may result in you missing a severe hypoglycemia (low blood glucose) or hyperglycemia (high blood glucose) occurrence and/or making a treatment decision that may result in injury. If your glucose alerts and readings from the Dexcom CGM do not match symptoms, use a blood glucose meter to make diabetes treatment decisions. Seek medical advice and attention when appropriate, including for any medical emergency.

Dexcom and Dexcom Clarity are registered trademarks of Dexcom, Inc. in the United States and/or other countries. ©2024 Dexcom, Inc. All rights reserved. MAT-5930